allard


Support for better life!

Instructions for Use

HyMo

Art.No. 39400-39411

SN

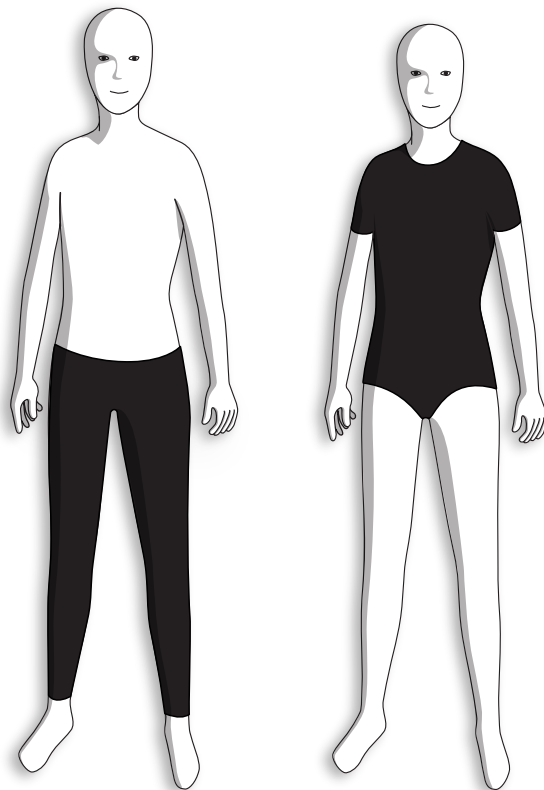Product Description

A made to measure textile orthosis for patients with a diagnosis within the hyper mobility spectrum. The orthoses are available as a body and/or trousers. This orthosis will provide light compression in order to improve postural control.

Intended Use

HyMo is an orthosis intended for children from 10 years old and adults suffering from hEDS (hypermobility Ehler Danlos Syndrom) or hypermobility within HSD (Hypermobility Spectrum Disorders). The orthosis can enhance proprioception and can improve joint stabilisation and in some cases reduce pain. The product is individually made and specified by the fitter to your needs. Single patient, multiple use.

Indications

hEDS – Ehlers Danlos Syndrom that presents with hypermobility, HSD (Hypermobility Spectrum Disorders)

Contraindications

Oedema, dislocated joints (without reduction), Vascular EDS (vEDS).

Limitations

When a new orthosis is introduced it is recommended to gradually increase the time of use until the patient has reached a usage of 8-10 hours a day. If worn at night then take a break for 2 hours the morning after. If used for pool training, a separate set of orthoses is recommended as it is important that the orthosis is clean and dry when used outside of the pool.

Warnings

Should not be used damp besides when used for pool training.

Check both orthosis and skin daily before usage. To ensure the best orthotic outcome, it is important that the orthosis is used daily.

Follow the instructions in the IFU/User Guide or as specified by your treating clinician. Consult with your physician whether to use Hymo or not during pregnancy.

HyMo provides a light compression and it is important to observe extra caution if the user suffers from skin problems, circulation disorders, has poor temperature regulation, has poor lung function or in case of pregnancy.

Material Specification

The fabric contains 59% Polyamide and 41% Elastane. Depending on model other materials are included. The product does not contain Latex. The product is not flame retardant treated.

The fabric is certified according to standard 100 by Oeko-tex, class 2 and is GRS-certificated.

Washing Instructions

Please use a perfume free mild detergent. We recommend that the HyMo orthosis is washed gently using a laundry bag. Close all zippers and Velcro straps before washing.

- Wash with similar colours.
- Do not use fabric conditioner.
- Do not dry in direct sunlight.
- Do not dry over the radiator.

Adjustments

Fitting of this product may only be performed by qualified healthcare professionals.

Questions

For any questions or issues, please contact your healthcare professional/fitter.

Information

Any serious incident that occurs in relation to the orthosis should be reported to the manufacturer and the competent authority of the Member State/country in which the

user and/or patient is established.

Disposal

Disposal in accordance with local legislation.

Advise for applying and wearing HyMo

Body

- Open all zippers and the closure at the crotch.
- Step into the garment and pull up over the bottom and the hips.
- Thread in one arm at a time and then pull up over the shoulders. Make sure the elbow panels are centered.
- Pull up the front (long) zipper first and then the rear (short) to get the shoulders in a good position.

Trousers

- Open all zippers.
- Step into the garment and pull each leg separately over your knees.
- Check that the knee panels are centered.
- Pull up the trousers to the waist and close with zipper and Velcro.
- Make sure the seams run straight along the sides of the legs.

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